

AirRise200

Quick User Guide



Charging

To ensure a long service life of the batteries, please ensure that the hoist is left on charge when not in use.

To charge the AirRise200™, follow the following procedure:

- Place the hand control in the wall charger (Figure 1)
- Check if the indicator light lights up (Figure 2)
- Leave hoist on charge for about 8 hours.

To ensure the batteries are kept in good condition, always put the hoist on charge after use. Never allow the battery to fully discharge.



Figure 1:
The wall charger



Figure 2:
Charging indicator light

Hand Control


The plug of the hand control should be placed firmly in the input at the bottom of the lift. The cord of the hand control can, if requested, be fitted with a strain release or fixed with screw-thread.

Top 2 buttons:
These control the hoist's lifting and lowering

Middle 2 buttons:
If the hoist is fitted with an electrically powered spreader bar, these buttons enable this to be used.

Bottom 2 buttons:
Turn the hoist on and off. When charging, the LED indicators light up at the bottom of the hoist. Should this be annoying, for example in a bedroom, this can be turned off with the red button. There is also the option of a bedroom mode, with which the charging indicators switch off automatically after 2 minutes.

Indicator light:
In the middle of the letter 'O' in INNOVA™, there is an LED indicator light. This tells you when the hoist is charging.

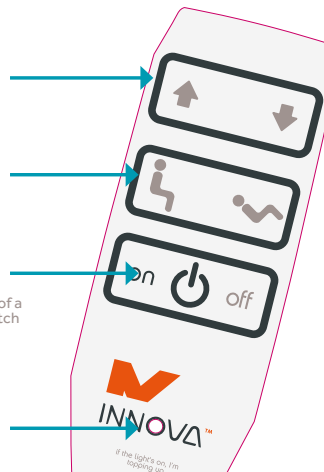

The most innovative
healthcare equipment in the world.

t | 0345 034 1450
f | 0345 034 1451
e | enquiries@innova.uk.com
www.innovacareconcepts.com

LED Indicator Lights



GREEN STATUS

If the hoist is active, the indicators light up green. The hoist switches itself off after approximately 30 minutes, the green light will then turn off. By pushing the On button, the lift will activate automatically. This is to save power.



ORANGE STATUS

The orange light indicates that the batteries should be charged at the first opportunity. The battery is not at critical level, but gives an indication that charging is desired. The batteries must be charged as described in section 2.3.



RED STATUS

The batteries have reached a critical level, the hoist must not be used, further discharge will damage the batteries. The batteries must be charged as described in section 2.3.

Weight limit

The weight limit of the AirRise200™ is 200kg (31.5 Stone).

Emergency Pull Cord

The hoist immediately stops all functions when the red emergency cord is pulled.

If you continue to pull the cord, the spreader bar will slowly move downwards. This movement stops as soon as the cord is released.

The AirRise200™ is now completely switched off. To re-activate the hoist, press the On button on the hand control.