



TR 4000/4200  
Bariatric shower trolleys

*Hygiene Equipment  
Designed for People*

Spare parts list

Version 2021-05-01



Registered at Medical Products Agency, Sweden as a MDR Class 1 product according to MDR 2017/745.

TR reserves the right to make design changes and/or improvements without notice.





---

## Table of contents

Introduction	Page 3
Manufacturer	Page 4
Spare parts list TR 4000/TR 4200	Page 5
Spare parts central brake/steering	Page 7
Spare parts, electric enclosures	Page 8
Wiring diagrams	Page 12



*Hygiene Equipment  
Designed for People*

---

## Introduction

Dear customer,

If you need spare parts to your bariatric shower trolley TR 4000/ TR 4200 and cannot find the part on following pages, contact TR Equipment or one of our qualified partners.



---

Manufacturer



TR Equipment AB  
Rundelgatan 2  
SE-573 35 Tranås  
Sweden

Phone: +46 (0)140 38 50 60  
[www.trequipment.com](http://www.trequipment.com)  
[info@trequipment.se](mailto:info@trequipment.se)

Group subsidiaries:

**USA**

TR Group Inc.

Phone: +1 (800) 7526900

[www.trequipment.com](http://www.trequipment.com)

[information@trequipment.com](mailto:information@trequipment.com)

**United Kingdom:**

TR Equipment Ltd

Phone: +44 (0) 844 335 8386

[www.trequipment.com](http://www.trequipment.com)

[info@trequipment.co.uk](mailto:info@trequipment.co.uk)

**Germany:**

TranspoRehamed GmbH

Phone: +49 (0) 212-645850-0


[www.transporehamed.de](http://www.transporehamed.de)

[info@transporehamed.de](mailto:info@transporehamed.de)



## Spare parts list

TR 4000/4200

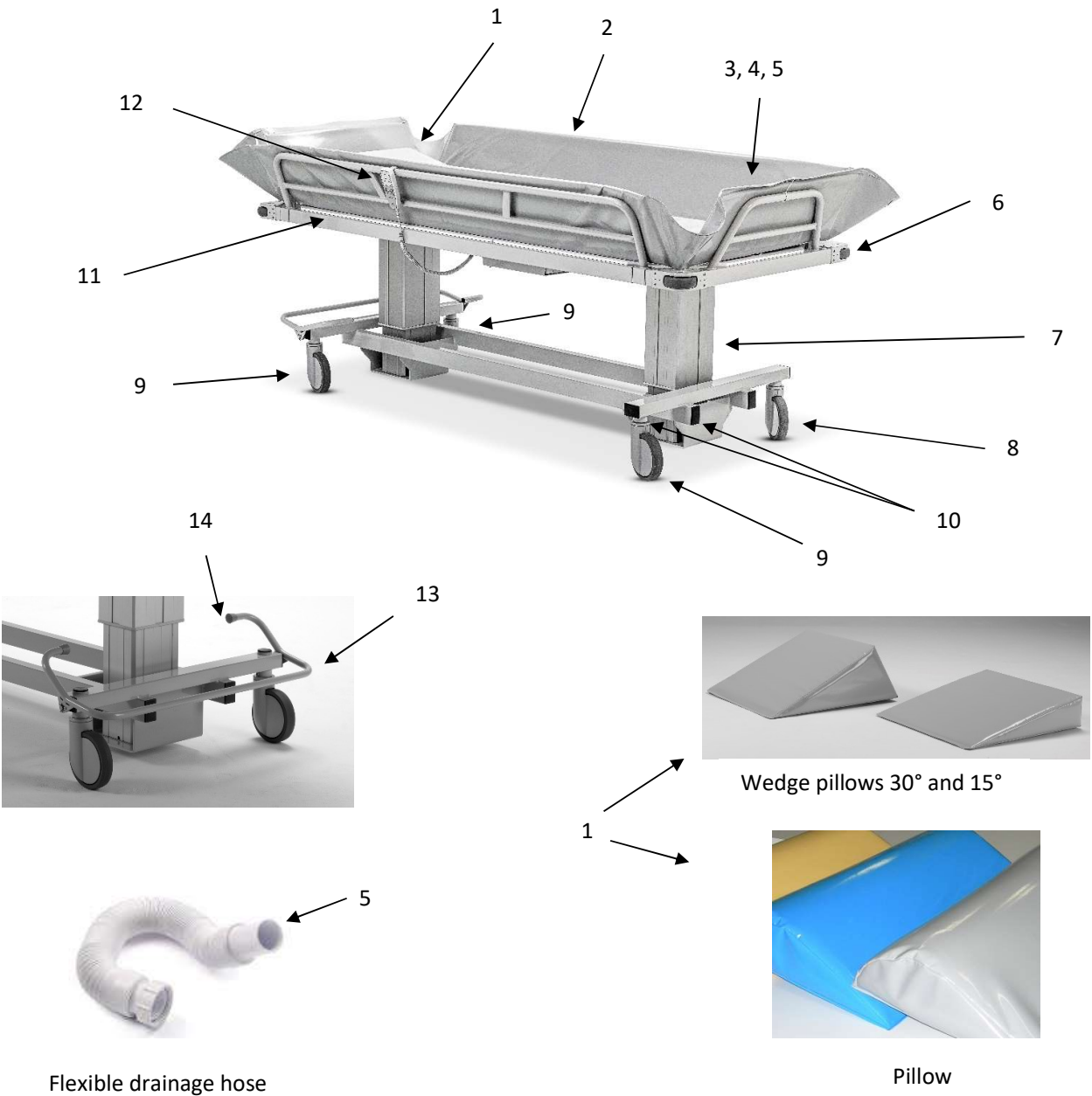



## Spare parts list

Position	Part.no	Spare part	Shower trolley
1	7418363	Wedge pillow Grey 15°	TR 4000, Option for TR4200
1	7418363-02	Wedge pillow Blue 15°	Option for TR 4000 / TR 4200
1	7418363-03	Wedge pillow Yellow 15°	Option for TR 4000 / TR 4200
1	7419488	Pillow	TR 4200
1	7419488-04	Pillow Blue	Option for TR 4200
1	7419488-07	Pillow Yellow	Option for TR 4200
1	7419491	Wedge pillow Grey 30°	Option for TR 4000 / TR 4200
2	7329504	Mattress Grey	TR 4000
2	7329504-02	Mattress Blue	Option for TR4000
2	7329913	Mattress Grey	TR 4200
2	7329913-02	Mattress Blue	Option for TR 4200
3	829392	Drainage	TR 4000 / TR 4200
4	829391	Drainage plug	TR 4000 / TR 4200
5	820818-4224	Flexible drainage hose 1M (std)	TR 4000 / TR 4200
5	820818-4224-25	Flexible drainage hose 2.5M (option)	TR 4000 / TR 4200
6	850133	Bumper wheel Ø100	TR 4000
6	850133	Bumper wheel Ø100	TR 4200 Serial number 1310-21494 & onwards
6	850133-75	Bumper wheel Ø75	TR 4200 Before serial number 1310-21494
7	819089	Lifting column	TR 4000 / TR 4200
8	850131	Straight steering castor	TR 4000 / TR 4200
9	850132	Castor with brake	TR 4000 / TR 4200
10	850575	End plug	TR 4000 / TR 4200
11	7329936	Plastic plate no.1 L 815mm	TR 4000
11	7329936-02	Plastic plate no.2 L 1225mm	TR 4000
11	7332411	Plastic plate L 2040mm (1 part)	TR 4200
12	819090-02	Hand control	TR 4000 / TR 4200 Serial number 0910-18179 up to 70461800.
12	7440000	Hand control	TR 4000 / TR 4200 Serial number 70461801 & onwards
13	3329982	Central brake/steering lever	TR 4000
13	3329982-02	Central brake/steering lever	TR 4200
14	3430052	End plug	TR 4000 / TR 4200
	3230018	Foldable side rail, long side	TR 4000
	3332387	Foldable side rail, short side (1pcs)	TR 4000
	3332393	Fixed side rail short side (1pcs)	TR 4000
	3232388	Foldable side rail, long side	TR 4200 Serial number 1310-21494 & onwards
	3230247	Foldable side rail, long side	TR 4200 Before serial number 1310-21494
	3432391	Fixed side rail, short side	TR 4200



## Spare parts list

TR 4000/4200




Position	Part.no	Spare part	Shower trolley
9	850132	Castor with brake	TR 4000 / TR 4200
13	3329982	Central brake/steering lever	TR 4000
13	3329982-02	Central brake/steering lever	TR 4200
15	3429977	Rod pipe	TR 4000
15	3429977-02	Rod pipe	TR 4200
16	831960-8050	Threaded rod	TR 4000 / TR 4200
17	850450	Fork incl. lock	TR 4000 / TR 4200
18	3429978	Link arm	TR 4000 / TR 4200
19	3429974	Hexagon rod, head end, L 862mm	TR 4000
19	3429974-02	Hexagon rod, foot end, L 772mm	TR 4000
19	3429974-03	Hexagon rod, head end, L 722mm	TR 4200
19	3429974-04	Hexagon rod, foot end, L 632mm	TR 4200
20	829106	O-ring	TR 4000 / TR 4200
21	3429979	Lever lock	TR 4000 / TR 4200



## Spare parts list

### TR 4000/4200

Electric enclosure, **part number 200118** from serial number 70461801 and onwards.

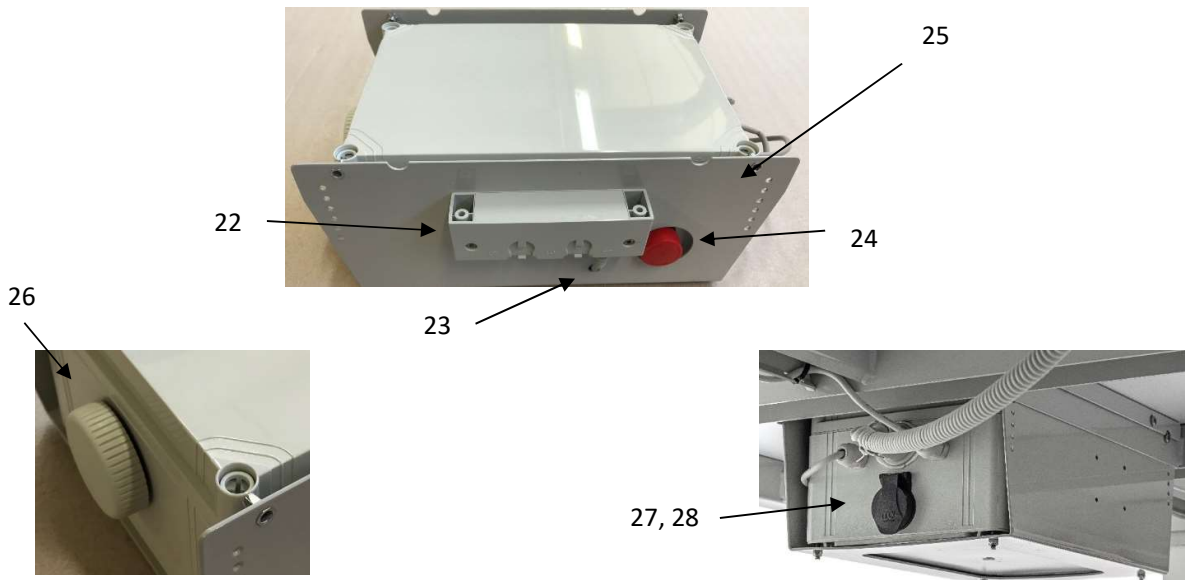

Position	Part.no	Spare part	Shower trolley
22	7440002	Junction box	TR 4000 / TR 4200
	7440004	Junction box plug (Mounted on the junction box)	TR 4000 / TR 4200
23	7440003	Junction box cable	TR 4000 / TR 4200
24	819310	Emergency stop	TR 4000 / TR 4200
	819312	Contact block for Emergency stop (Inside the enclosure)	TR 4000 / TR 4200
	829144	EPDM gasket for Emergency stop	TR 4000 / TR 4200
25	3329960	Mounting plate for enclosure	TR 4000 / TR 4200
26	819171	Ventilation plug	TR 4000 / TR 4200
27	819150	Mag code Port cover	TR 4000 / TR 4200
28	819148	Mag code connector	TR 4000 / TR 4200
	829143	EPDM gasket for Mag Code port	TR 4000 / TR 4200
	851650	Cable glands PG9	TR 4000 / TR 4200
	851650-03	EPDM gasket for PG9	TR 4000 / TR 4200
		<b>Inside the enclosure:</b>	
	819030	12V 7Ah Lead battery (2pcs)	TR 4000 / TR 4200
	819354-02	Circuit board	TR 4000 / TR 4200
	7429679	Cabling incl. Fuse and fuse holder	TR 4000 / TR 4200
		<b>Battery charger</b>	
	0541226	Battery charger Mag Code	TR 4000 / TR 4200
	819140	EU Plug for battery charger	TR 4000 / TR 4200
	819141	US Plug for battery charger	TR 4000 / TR 4200
	819142	UK Plug for batter charger	TR 4000 / TR 4200





*Hygiene Equipment  
Designed for People*



## Spare parts list

### TR 4000/4200

Electric enclosure, **part number 200111** from serial number 1401-216595 to 70461801 (70461801 not included).



27, 28

29



26

Position	Part.no	Spare part	Shower trolley
24	819310	Emergency stop	TR 4000 / TR 4200
	819312	Contact block for Emergency stop (Inside the enclosure)	TR 4000 / TR 4200
	829144	EPDM gasket for Emergency stop	TR 4000 / TR 4200
25	3430021	Mounting plate for enclosure	TR 4000 / TR 4200
26	819171	Ventilation plug	TR 4000 / TR 4200
27	819150	Mag code Port cover	TR 4000 / TR 4200
28	819148	Mag code connector	TR 4000 / TR 4200
	829143	EPDM gasket for Mag Code port	TR 4000 / TR 4200
	819097	Female connector for handcontrol	TR 4000 / TR 4200
29	851650	Cable glands PG9	TR 4000 / TR 4200
	<b>Inside the enclosure:</b>		
	819030	12V 7Ah Lead battery (2pcs)	TR 4000 / TR 4200
	819356-2 / *819354-02	Circuit board	TR 4000 / TR 4200
	7429679	Cabling incl. Fuse and fuse holder	TR 4000 / TR 4200
<b>Battery charger</b>			
	0541226	Battery charger Mag Code	TR 4000 / TR 4200
	819140	EU Plug for battery charger	TR 4000 / TR 4200
	819141	US Plug for battery charger	TR 4000 / TR 4200
	819142	UK Plug for batter charger	TR 4000 / TR 4200


\*819354-02 will replace 819356-2



## Spare parts list

### TR 4000/4200

Electric enclosure, from serial number 0910-18179 to 1401-21695 (1401-21695 not included).

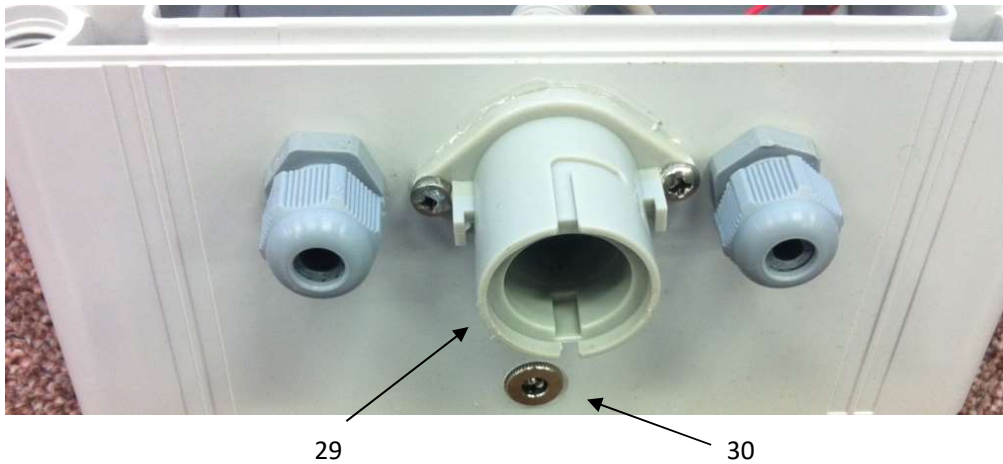

Position	Part.no	Spare part	Shower trolley
29	819097	Female connector for handcontrol	TR 4000 / TR 4200
30	819434	Connector for battery charger	TR 4000 / TR 4200
30	7429678	Cable for connector (Black)	TR 4000 / TR 4200
30	7429678P	Cable for connector (Orange)	TR 4000 / TR 4200
	851650	Cable glands PG9	TR 4000 / TR 4200
		<b>Inside the enclosure:</b>	
	819030	12V 7Ah Lead battery (2pcs)	TR 4000 / TR 4200
	819356-2 / *819354-02	Circuit board	TR 4000 / TR 4200
	7429679	Cabling incl. Fuse and fuse holder	TR 4000 / TR 4200
		<b>Battery charger</b>	
	0541226	Battery charger Mag Code	TR 4000 / TR 4200
	819140	EU Plug for battery charger	TR 4000 / TR 4200
	819141	US Plug for battery charger	TR 4000 / TR 4200
	819142	UK Plug for batter charger	TR 4000 / TR 4200


\*819354-02 will replace 819356-2



## Spare parts list

### TR 4000/4200

Electric enclosure, serial numbers before 0910-18179.

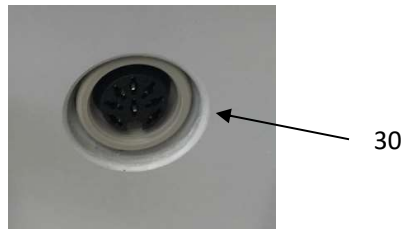

Position	Part.no	Spare part	Shower trolley
31	7900200	Female connector for handcontrol	TR 4000 / TR 4200
	851650	Cable glands PG9	TR 4000 / TR 4200
		<b>Inside the enclosure:</b>	
	819030	12V 7Ah Lead battery (2pcs)	TR 4000 / TR 4200
	819356-2 / *819354-02	Circuit board	TR 4000 / TR 4200
	7429679	Cabling incl. Fuse and fuse holder	TR 4000 / TR 4200

\*819354-02 will replace 819356-2




## Spare parts list

### Circuit board 819354-02

Circuit board 819354-02 will replace 819356-2. Circuit board 819354-02 will fit all previous TR 4000 and TR4200 models.

See wiring diagram below for TR 4000/TR 4200 and for serial numbers from 70461801 and onwards.

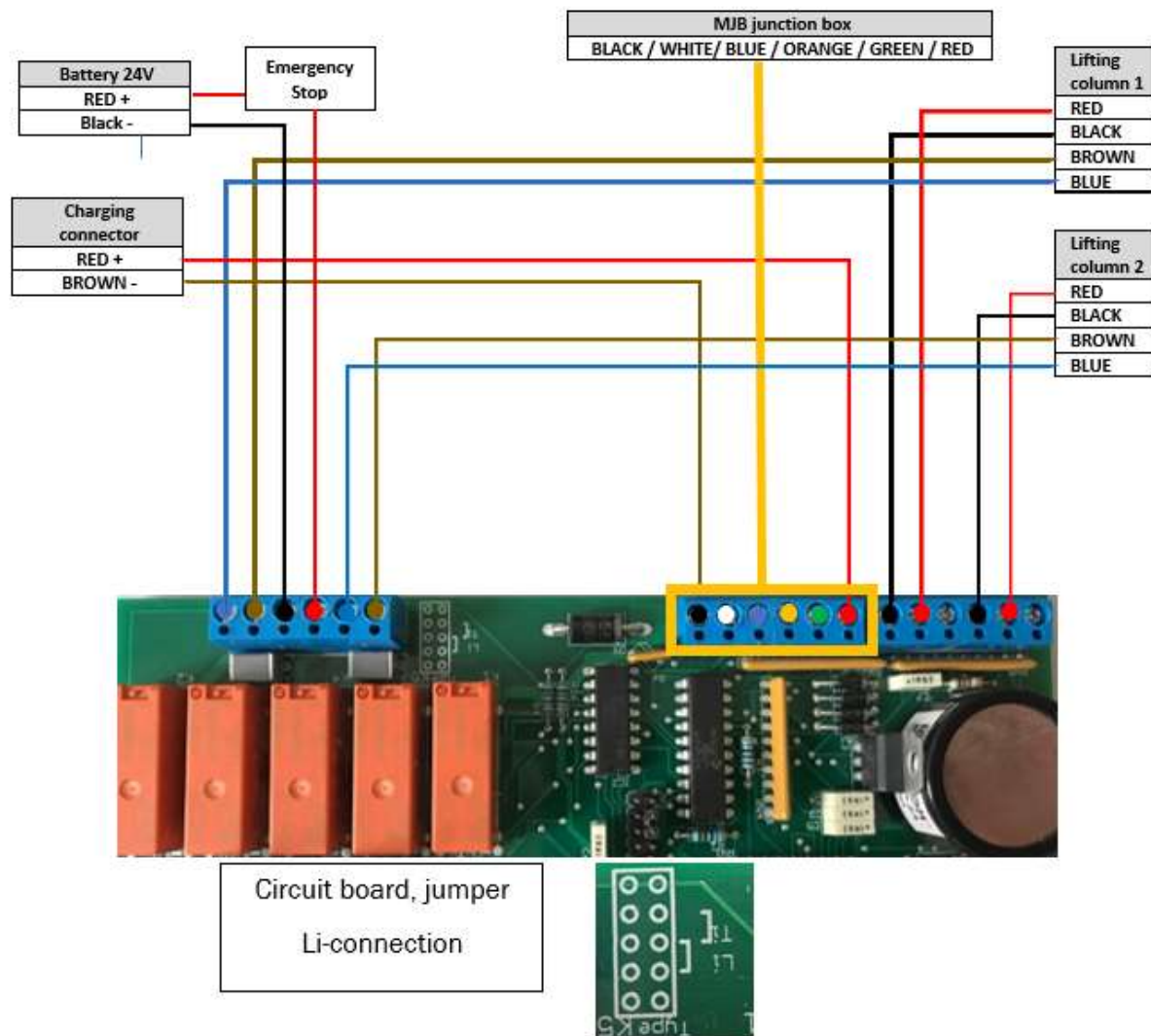



## Spare parts list

### Circuit board 819354-02

Circuit board 819354-02 will replace 819356-2. Circuit board 819354-02 will fit all previous TR 4000 and TR4200 models.

See wiring diagram for TR 4000/TR 4200 with serial numbers prior to 70461801.

